

Klomipramin	Isoxsuprin	Fenobarbital	D,L-Tryptofan
Klonidin	Ketamin	Fentermin	Tyramin
Kokain hydrochlorid	Ketoprofen	L-fenylefrin	Kys. močová
Kortison	Labetalol	β-fenylethylamin	Verapamil
(-) kotinin	Loperamid		Zomepirak

<LITERATURA>

1. Moolchan, E., et al, "Saliva and Plasma Testing for Drugs of Abuse: Comparison of the Disposition and Pharmacological Effects of Cocaine", Addiction Research Center, IRP, NIDA, NIH, Baltimore, MD. As presented at the SOFT-TIAFT meeting October 1998.
2. Baselt RC. Disposition of Toxic Drugs and Chemicals in Man. 2nd Ed. Biomedical Publ., Davis, CA. 1982; 488

<SEZNAM SYMBOLŮ>

	Prostudujte pokyny k použití		Testů na kit		Autorizovaný zástupce
	Pouze pro <i>in vitro</i> diagnostické použití		Použijte do		Nepoužívat opakované
 2°C / 30°C	Skladujte mezi 2-30°C		Číslo šarže		Katalogové číslo
	Nepoužívejte, pokud je obal poškozen		Výrobce		

 **Hangzhou AllTest Biotech Co.,Ltd.**
 #550, Yinhai Street
 Hangzhou Economic & Technological Development Area
 Hangzhou, 310018 P.R. China
 Web: www.alltests.com.cn Email: info@alltests.com.cn





MedNet EC-REP GmbH
 Borkstrasse 10,
 48163 Muenster,
 Germany

Číslo: 145483303
 Datum revize: 2023-11-06